




# Hear Clear Fitting Tip

## Hear Clear

Starkey has made improvements to the Hear Clear wax prevention system. First, the new Hear Clear is now coated with an oleophobic and hydrophobic material to resist not only water but also oils. This helps prevent wax from entering the receiver and improves patient satisfaction by reducing wax-related repairs. Second, the method of replacing the Hear Clear has been changed. The new instructions are found on the inside flap of every Hear Clear package. New Hear Clear sticks can easily be identified by the red color and will work in all existing Hear Clear receptacles.

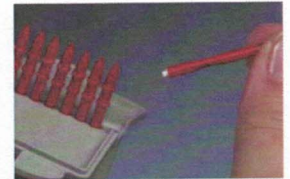

## Changing Hear Clear wax protector

With a new Hear Clear stick follow the steps below to remove and insert a Hear Clear wax protector:

### Removal

| Step 1                   | Step 2  | Step 3                             |
|--------------------------|---|------------------------------------|
|                          |   |                                    |
| Using double-notched end | Push & turn Hear Clear stick into existing Hear Clear | Pull Hear Clear stick straight out |

### Insertion

| Step 1               | Step 2   | Step 3                             |
|----------------------|--|------------------------------------|
|                      |  |                                    |
| Using un-notched end | Insert Hear Clear straight into hearing aid receptacle | Pull Hear Clear stick straight out |